





Here carefully loosen in the direction of the arrow

How to use the FecesCatcher

Dear user,

The FecesCatcher facilitates the taking of a stool sample. It is simple and hygienic. For reliable results, without the risk of contaminating the sample. Water, urine, detergents and deodorants may affect the outcome of the analysis of the sample.

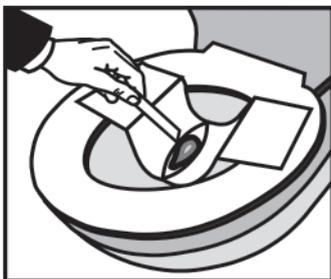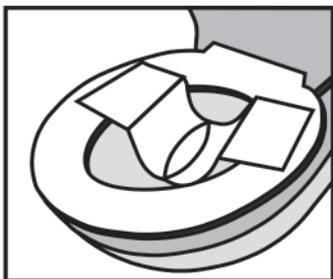

You should use the FecesCatcher as follows:

1. Open the FecesCatcher carefully in the direction of the arrows.
2. Paste the FecesCatcher on the back of the toilet seat, as shown in the second image.
3. Avoid that the FecesCatcher comes in contact with water.
4. After the defecating, take the sample as indicated in the instructions that came with the test set.



Here carefully loosen in the direction of the arrow

Carefully loosen in the direction of the arrow, here



5. After taking the sample loosen the FecesCatcher on both ends and squeeze the ends together.
6. Drop the FecesCatcher in the toilet bowl and wait a moment, until the paper is soft and can easily be flushed down the toilet.

The FecesCatcher should also be used in shelf toilets, because detergents or fragrances contained in the flush water may adversely affect test results.

IsohelixTM
DNA Sampling and Purification

© Copyright Isohelix
www.isohelex.com
info@isohelex.com

Carefully loosen in the direction of the arrow, here

