Instructions for Use: Isohelix™ StoolFix 10ml collection tubes, pre-filled with 1.5ml Stabilisation Buffer, Using Isohelix Swabs

REF STF/K/15

IVD **CE** **UK** **CA**

Intended Use: For the collection and stabilisation of Microbiome DNA from human faecal samples delivered into a sealable tube pre-filled with a storage solution in view of downstream microbiome DNA applications.

Kit Contents: 1 x 10ml collection tube containing 1.5ml of StoolFix stabilisation buffer.

Materials required but not supplied: Isohelix Collection Swabs (SK-3S/MS-02).


Storage and Stability: Store at Room Temperature pre and post collection (15°C-30°C). Microbial profile stability on stool swabs stored in StoolFix tubes at Room Temperature is maintained for up to 2 months. Store samples upright following collection.

Warnings and Precautions: FOR EXTERNAL USE ONLY, DO NOT ingest stabilising solution. If the stabilising solution comes into contact with the skin or eyes, wash with plenty of water. Cap may be a choking hazard to small children. In case of any serious issues contact Isohelix and local competent authorities.

MSDS and other language collection Instructions available at www.isohelex.com

Before proceeding with swab collection: Prepare and collect a faecal sample using an appropriate stool catcher kit or container.


- 1

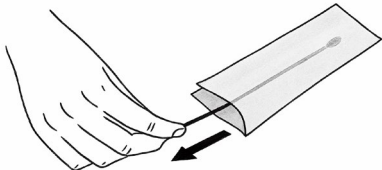
Pull open the collection swab wrapper package from one end. Remove the swab from the wrapper, taking care not to touch the white swab head with your fingers.
- 2

Using **both** sides of the swab, brush the surface of the faecal sample until there is good coverage of sample collected onto the swab. Take care not to overload the swab with sample.
- 3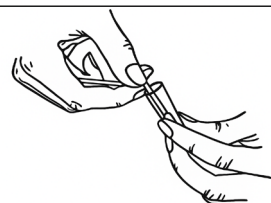

Break off the swab head by pressing the breakpoint of the swab stick against the tube's rim, bending it to one side until it snaps, allowing the swab head to drop into the collection tube with the solution.
- 4

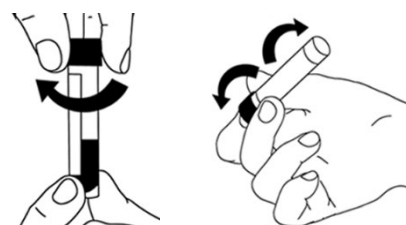Replace the tube screw cap, tightening to ensure the tube is securely sealed. Shake the tube for ten seconds to mix the sample with the solution. Dispose of the remaining faecal sample and wash hands well following sample collection.

Label Legend:

IVD	In vitro diagnostic	CE	CE Marking
UK	UKCA marking	LOT	Lot number
	Consult instructions	15°C / 25°C	Storage instructions
	Do not re-use	REF	Catalog Number
	Use by date		
EC REP	Authorised European Representative		

Warning:
H315
H319

Note: It is the sole responsibility of the Laboratory or Professional User to verify and correctly validate the use, technically and legally, of this StoolFix kit.

Cell Projects Ltd
2 Roebuck Business Park
Ashford Road, Harrietsham,
Kent, ME17 1AB, UK

Advena Ltd
Tower Business Centre,
2nd Flr., Tower Street,
Swatar, BKR 4013 Malta

Isohelix is a division of Cell Projects

For swab or DNA isolation queries email: info@isohelex.com
www.isohelex.com
For Molecular Biology Solutions www.cellprojects.com

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